

QUICK START GUIDE English

For a perfect start.



www.miyoworld.eu



PREPARATION & INSTRUCTIONS FOR USE



Preparation

- Before applying the material, the surface must be clean and free of surface contamination.
- The surface can be sandblasted with 50 µm and 2 bar pressure.

Instructions for use

- Do not pour surplus liquid out of the container. Mix it back into a paste.
- Mix MiYO Color, MiYO Structure and Glaze Paste thoroughly with a metal free spatula before each use.
- The pastes must not come into contact with water.
- Always use a clean and dry brush.

Application

- Apply material to the prepared surface according to its purpose and required effect.

Storage

- Dry between +12°C to +38°C.

**MIX MIYO COLOR, STRUCTURE AND
GLAZE PASTE THOROUGHLY WITH A
METAL-FREE SPATULA BEFORE
EACH USE.**



**Find more information in
YOUR LANGUAGE
on the USB Stick**



FIRING CHARTS & FIRING GRAPHS



Firing parameter*	COLOR STRUCTURE GLAZE PASTE FIRING			
	ZrO ₂	InSync Zr	Lithium Disilicate	Press to Zirkonia
Start temperature [°C]	400 - 450	400 - 450	400 - 450	400 - 450
Dry time [min]	3	3	3	3
Closing time [min]	4	4	4	4
Heat rate 1 and 2 [°C/min]	45	45	45	45
Holding time 1 [s]	30 - 45	30 - 45	30 - 45	30 - 45
Vacuum start [°C]	580	580	580	580
End temperature [°C]	720	720	710	720
Vacuum end [°C]	720	720	710	720
Holding time 2 [s] **	30 - 60	30 - 60	30 - 60	30 - 60
Opening time [min]	1	1	1	1

* The firing temperatures are standard values and may vary depending on the type of furnace.

** Depending on the desired shine, holding time should be 30 - 60 seconds.

HT STRUCTURE FIRING	
ZrO ₂	InSync Zr
400 - 450	400 - 450
3	3
4	4
45	45
30 - 45	30 - 45
600	600
775	775
775	775
60	60
1	1

Find more information in the customer portal:





www.miyoworld.eu/en/login-en/kundenlogin



Firing graphs

Adjustment of different firing parameters, according to the size of the restoration, for details see Workbook p.17. **Note: Delayed vacuum start!**

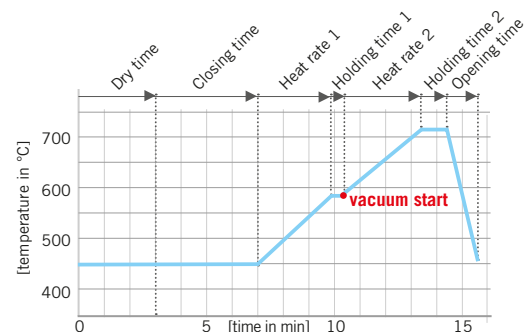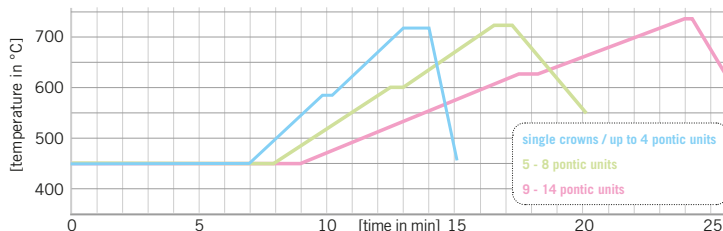

TRAINING & SUPPORT



Training videos, furnace programming ...

In the customer portal we provide you with exclusive training and information material. Learn how to use MiYO and MiYO pink safely with the help of the training videos - available in different languages. Firing instructions and notes on furnace programming help you to program your furnace.

Register: www.miyoworld.eu/en/login-en/registrierung

To register you will need the MiYO Kit number, this can be found on the back of this Quick Start Guide and in the MiYO kit under the liquid.



MiYO Online Training

Check out everything about MiYO or MiYO pink with a practical online demo. Our application technicians will give you an overview of the system, show you how to design an anterior or gingiva with MiYO and answer your questions about MiYO and MiYO pink. The online demos are suitable for newcomers and MiYO users. Register free of charge ...

www.miyoworld.eu/en/training/online



Instruction manuals

All documents concerning MiYO, MiYO pink and the glaze pastes are available in different languages for you to download from our website or on the enclosed USB stick

www.miyoworld.eu/en/downloads



Support

Do you have any questions about MiYO?
Please get in touch with us.

E-Mail: miyo@jensendental.de
Phone: +49 (0) 7123 92260
www.miyoworld.eu/en/contact



Individual training & courses






















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


For an individual MiYO training and MiYO courses please contact your local MiYO dealer.

www.miyoworld.eu/en/dealer



PRICE LISTS & ORDER INFORMATION


















MiYO

Article		Art.no.	Price €
TRANSLUCENT 4g			
Trans Shade A		0877	43,00
Trans Shade B		0878	
Trans Shade C		0879	
Trans Shade D		0880	
Trans Straw Fluor		0862	
Trans Sage Fluor		0893	
Trans Lotus Fluor		0861	
Trans Clementine Fluor		0863	
Trans Storm Fluor		0860	
Trans Cobalt Fluor		0894	
Trans Smoke Fluor		0875	
Trans Slate Fluor		0895	
Trans Lumin Fluor		0876	
Trans Lumin Plus Fluor		0896	
Trans Sunflower Fluor		0950	
Trans Garnet		0952	
MAMELON 4g			
Mamelon Wheat Fluor		0866	43,00
Mamelon Coral Fluor		0867	
Mamelon Pumpkin Fluor		0868	
HALO 4g			
Halo Spring Fluor		0864	43,00
Halo Autumn Fluor		0865	
COLOR 4g			
Snow Fluor		0857	43,00
Linen Fluor		0858	
Fissure Fluor (Pulver)		0859	

Article		Art.no.	Price €
STRUCTURE 4g			
Structure Window Fluor		0869	48,00
Structure Ghost Fluor		0873	
Structure Enamel Fluor		0872	
Structure Ice Fluor		0870	
Structure Blush Fluor		0871	
HT STRUCTURE 4g			
HT Structure Enamel Fl.		0953	48,00
HT Structure Window Fl.		0954	
GLAZE PASTE 4g			
InSync Glaze Paste Fluor		0819	39,00
STAIN / GLAZE LIQUID INSYNC one-for-all			
Stain / Glaze Liquid 25ml		0800/25	11,60
Stain / Glaze Liquid 50ml		0800/50	23,00
MIYO Kit			
18 x MiYO Color 5 x MiYO Structure 1 Glaze Paste 1 Stain / Glaze Liquid 25 ml		0881- MiYO Kit	990,00
MIYO Kit Color+			
22 x MiYO Color 2 x MiYO Structure 1 Glaze Paste 1 Stain / Glaze Liquid 25 ml		0881- MiYO Kit Color+	990,00

The price list is valid from 01.04.2021.
All prices plus VAT and shipping costs.
Errors and omissions excepted. The
General Terms and Conditions of Jensen GmbH apply.

MiYO pink

Article		Art.no.	Price €
TRANSLUCENT 4g			
Trans Raspberry		0887	43,00
Trans Copper		0888	
Trans Midnight		0883	
Trans Carnation		0951	
Trans Garnet		0952	
COLOR 4g			
Flamingo		0881	43,00
Crimson		0882	
Plum		0884	
Merlot		0885	
Sorbet		0886	
Salmon		0889	
Sable		0897	
Thistle		0898	
Venule (Pulver)		0957	
STRUCTURE 4g			
Structure Orchid		0890	48,00
Structure Rouge		0891	
Structure Frost		0892	
GLAZE PASTE 4g			
MiYO Glaze Paste No Fluor		0821	39,00
STAIN / GLAZE LIQUID INSYNC one-for-all			
Stain / Glaze Liquid 25ml		0800/25	11,60
Stain / Glaze Liquid 50ml		0800/50	23,00
MiYO pink Kit			
11 x MiYO pink Color 3 x MiYO pink Structure 1 MiYO Glaze Paste No Fluor 1 Stain/Glaze Liquid 25 ml		0881- MiYO pink	599,00



Register for the customer portal and get more information

As a MiYO customer, you have exclusive access to the customer portal.
To do so, please register at www.miyoworld.eu under LOGIN > REGISTRATION.
To register, you need your MiYO Kit number.

Your MiYO Kit number



<https://www.miyoworld.eu/en/login-en/registrierung>

Manufactured by:

Chemichl AG
Landstrasse 114
9490 Vaduz, Liechtenstein
info@chemichl.com



Sales and Technical Support Europe:

Jensen GmbH
Gustav-Werner-Straße 1
72555 Metzingen, Germany

Telefon 0800-857 32 30
info@jensendental.de
www.miyoworld.eu
www.jensendental.de

... more MiYO

