

Sales and Technical Support Europe:

Jensen GmbH
Gustav-Werner-Straße 1
72555 Metzingen, Germany
0800-857 32 30 | +49 7123-92260
info@jensendental.de
www.jensendental.de | www.miyoworld.eu

Manufactured by:

Chemichl AG
Landstrasse 114
9490 Vaduz, Liechtenstein
info@chemichl.com
www.chemichl.com



4366040523



QUICK START GUIDE English

For a perfect start.



! Preparation | Instructions for use

Preparation

- Before applying the material, the surface must be clean and free of surface contamination.
- The surface can be sandblasted with 50/110 µm and 1.5 bar pressure.

Instructions for use

- Do not pour surplus liquid out of the container. Mix it back into a paste.
- Mix MiYO Color, MiYO Structure and Glaze Paste thoroughly with a metal free spatula before each use.
- The pastes must not come into contact with water.
- Always use a clean and dry brush.

Storage and disposal

- Store in a dry place.
- Small quantities can be deposited in household waste.



Mix thoroughly with a metal-free spatula.

🔥 Firing chart

Firing parameter*	
Start temperature [°C]	400 - 450
Dry time [min]	3
Closing time [min]	4
Heat rate [°C / min]	45
Holding time 1 [s]	30 - 45
Vacuum start [°C]	580
End temperature [°C]	710 - 720
Vacuum end [°C]	710 - 720
Holding time 2 [s]	30 - 60
Opening time [min]	1

* The firing temperatures are standard values and may vary depending on the type of furnace.

📄 Downloads

You can find further documents for download at:
www.jensendental.de/en/downloads-en



🛒 Webshop

24/7 you can order in our order in our webshop.
<https://shop.jensendental.de/>



💬 Support

General requests:
info@jensendental.de
+49 7123 92260
Technical support:
support@jensendental.de

